



# AKRON® M6M Model 8462

# **USER MANUAL**

#### AKRON 8462

# **Contents**

1.	Introduction	2
	Warnings, Cautions	2 2
	General Warnings	2
2.	Installation	3
3.	Operation	<b>4</b> 5 6
4.	Accessories – Operating Instructions Cyriax Horns Hole & Horn Lean Back Frame Fixation Rails	<b>7</b> 7 7 7 7
5.	Cleaning and Decontamination	8
6.	Troubleshooting	8
7.	Maintenance	9
8.	Warranty and Service	<b>10</b>
9.	<b>Revision Status</b>	11
10.	<b>Technical Specification</b>	12

CE

"The **CE** mark on this product demonstrates conformity with the EC Directive 93/42/EEC."

#### 1. Introduction

Always keep this Operating Manual available for reference.

# **Warnings & Cautions**

#### **WARNINGS**

Warnings identify possible hazards in procedures or conditions which, if not correctly followed, could result in death, injury or other serious adverse reactions.

#### **CAUTIONS**

Cautions identify conditions or procedures which, if not correctly followed, could result in equipment failure or damage.

#### **General Warnings**

#### **WARNINGS**

Use only accessories that have been designed or approved for use with this couch.

Do not use electric couches in the presence of flammable gasses such as an anaesthetic agent.

Maximum Load 180 (396 lbs) distributed evenly.

Do not concentrate weight on either side of the couch.

Plug acts as a disconnect device.


Do not allow children to play with the couch at any time.

Adequately supervise children in the proximity of the couch.

# 2. Installation

# **Adjustable Foot**

# WARNING Lock castors before adjusting.

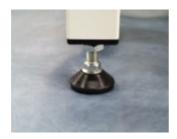

- A) Lock castors first.
- B) Adjust foot.
- C) Lock foot by tightening half nut.


#### WARNING

Do not insert the 2-pin mains connector directly into a 3-pin 13A mains outlet.

Always use the adapter provided.

Plug acts as a disconnect device.

#### **Electric Models**

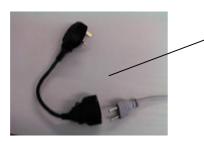

Use adapter if 3 pin 13A plug is required. (Order reference 5S0409Z)

Ensure mains supply cable is not stretched when connected.

Ensure, during couch use, no obstacles are in its path.

Ensure hand / foot switch tubes or leads are not trapped by moving parts of the couch.

#### CAUTION

When not in use, switch couch off.

# 3. Operation

# **Elevation Controls**

#### WARNING

Safely position patient before elevating couch.
Position all electrical cables away from the moving parts.
Position Hand/Foot switch where accidental operation is not possible.

# **Electric Models**



Handswitch

Press **♦** to lower couch.



Footswitch

AUF
Press ↑ to raise couch

# **Foot Bar Switch**



Press tube done to raise couch

Lift tube up to lower couch

# 3. Operation

# **Wheel Locking**


#### **WARNING**

Lock couch wheels/castors prior to patient transfer or treatment. The retractable castors are not designed for transportation of patients. Disconnect and coil electrical cord away from moving parts prior to repositioning couch.

# **Castors**



Castors Unlocked

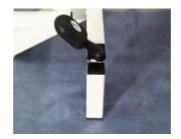

Castors Locked

# **Head & Foot Sections**

#### **CAUTION**

Support upholstery section while manually altering the angle.

#### Head Section Operating Lever



- A) Press lever UP to alter section angle.
- B) Support section when altering angle.
- C) Release lever to lock section in position.

#### **Head Section Arm Rests**



- A) Tilt armrest towards head section.
- B) Move armrest into position.
- C) Release armrest to lock into position.

# 3. Operation

#### **Foot Section**

#### **CAUTION**

The foot section is able to move in two independent modes. It is recommended that Operators familiarise themselves with the functions of each mode, prior to using the couch with a patient.

# **Flexion & Extension**



Foot Section Operating Lever

To adjust the foot section for flexion & extension :-

- A) Rotate the lever positioned under the foot section, clockwise or anti-clockwise.
- B) Raise or lower section to desired angle.
- C) To secure foot section at correct angle, continue to rotate the lever.

Foot Section Angles =  $+26^{\circ}$  -25°

# **Side Bending**



Side Bending Release Lever

To release the foot section to enable side bending:-

- A) Rotate the lever under the mid section upholstery anti-clockwise.
- B) Rotate the foot section to desired position.
- C) Rotate the lever clockwise to secure foot section in desired position.

Angles =  $30-35^{\circ}$  along the horizontal axis.

# **Breathing Hole and Plug**



Remove plug for patient prone position.

Press out from underneath.

# **4.** Accessories – Operating Instructions

# **Cyriax Horns**



Adjust Cyriax Horns to suit patient by :-

- A) Unlocking the locking knobs.
- B) Adjust position of horns.
- C) Tighten locking knobs.

# Hole & Horn



To fit Horn into Hole:-

- A) Simply locate horn into hole.
- B) Ensure horn is fully inserted prior to use.

# **Lean Back Frame**



To adjust Lean Back Frame:-

- A) Remove locating pins from frame.
- B) Slide frame to required position.
- C) Replace pins into holes at the end of the frame.

# **Fixation Rails**



Ensure straps are correctly located through fixation rails before use.

# 5. Cleaning and Decontamination

#### WARNING

Do not use Alkalis or other organic solvents for routine cleaning.

Replace torn or damaged vinyl immediately.

Always cover vinyl with fresh paper or sheet between patients.

If contamination is known or suspected always decontaminate couch before use.

Frequency of routine couch cleaning will depend on exact application.

#### **CLEANING / DECONTAMINATION**

Wear suitable protective clothing

Disconnect mains supply

Locate couch in a suitable cleaning area with castors disengaged or wheels locked.

- A) Using a clean damp cloth, warm water and soap carefully wipe couch from top to bottom, taking care around electrical components.
- B) Wipe couch carefully from top to bottom using a damp cloth and clean water.
- C) Wipe couch carefully dry with clean paper towels from top to bottom.

If **DECONTAMINATING** then repeat steps a) to c) using a dilute Hypochlorite / warm water mixture (10,000ppm) instead of warm water and soap.

Safely discard cleaning / decontaminating materials.

Wash hands thoroughly.

Put couch back into service.

# 6. Troubleshooting - Electric Actuator

Symptom	Possible cause	Action
Actuator will not work	Power disconnected	Ensure mains supply is connected
	Fuse blown	Replace fuse in socket
	Hand/footswitch disconnected	Check that the hand/foot switch is fully inserted into the actuator
Actuator runs when Bar	Out of balance system	Vent air tube by disconnecting from
is not pressed	out or outlined system	actuator, then reconnect
(Foot Bar Switch)		

If the problem persists consult your local Akron Supplier.

#### 7. Maintenance

Check couch visually before use.

Carry out routine servicing every six months or more frequently if in heavy use.

Routine servicing or repair must only be performed by suitably qualified person.

#### **WARNING**

Before commencing any maintenance activity, disconnect the couch from the electrical supply.

Plug acts as disconnect device.

Examine the couch for any signs of damage. Make sure that all nuts, bolts and other fasteners are tight. Do not over tighten as this may cause binding.

Upholstery fasteners should be checked, tightened or replaced if necessary.

Check section hinge pins to ensure that no pin protrudes.

Check gas strut controls, throughout the full range of movement for locking and assistance.

Check hydraulic couches for smooth operation and signs of leakage.

#### WARNING

All electrical checks should be performed by a suitably qualified and licensed electrician.

Neither BLUE or BROWN wires are to be connected to the earth terminal of a three pin plug.

Examine the flexible cables for cuts, abrasions, kinking or other deterioration and replace if necessary.

Ensure that the power supply connector plug is securely attached.

Check mains plug connections are tight and fuse is correctly rated (5A).

# 8. Warranty and Service (UK only)

Huntleigh Akron's standard terms and conditions apply to this couch. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service or maintenance please contact:

Service Department Huntleigh Healthcare 310-312 Dallow Road

Luton

Beds Telephone: 01582 745700 LU1 1TD Fax: 01582 745869

For information on this product, or any other Huntleigh Akron product, please contact:

Huntleigh Akron 1 Farthing Road

Ipswich Telephone: 01473 461042 IP1 5AP Fax: 01473 462924

Huntleigh Akron T/A a Division of Huntleigh Nesbit Evans Limited A Member of the Huntleigh Technology PLC Group of Companies © Huntleigh Technology PLC 2001.

If for any reason your couch is being returned, please inform Huntleigh Akron prior to sending, and:

- 1. Clean the product, as described in the cleaning section.
- 2. Pack it in suitable packing.
- 3. Attach the decontamination certificate (or other written statement declaring that the product has been cleaned) to the outside of the package.
- 4. Mark the package "Service Department".

Manufactured in the UK by Huntleigh Akron. As part of the ongoing development programme this company reserves the right to modify specifications and materials of this couch without notice.

# 8. Warranty and Service (UK only)

Huntleigh Akron's standard terms and conditions apply to this couch. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance and any questions regarding this, or any other Huntleigh Akron Product, please contact:

Huntleigh Akron

1 Farthing Road

Ipswich

Suffolk

IP1 5AP.

England

Telephone: +44 (0) 1473 461042 Fax: +44 (0) 1473 462924

Huntleigh Akron T/A a Division of Huntleigh Nesbit Evans Limited

A Member of the Huntleigh Technology PLC Group of Companies

© Huntleigh Technology PLC 2001.

If for any reason your couch is being returned, please inform Huntleigh Akron prior to sending and:

- 1. Clean the product, as described in the cleaning section.
- 2. Pack it in suitable packing.
- 3. Attach the decontamination certificate (or other written statement declaring that the product has been cleaned) to the outside of the package.
- 4. Mark the package "Service Department".

Manufactured in the UK by Huntleigh Akron. As part of the ongoing development programme this company reserves the right to modify specifications and materials of this couch without notice.

#### 9. Revision Status

8462					
PAGES	ISSUE	WRITTEN	DATE		
ALL PAGES	UM - 01	S.Rees	08/06/98		
ALL PAGES	UM - 02	S.Rees	02/12/98		
PAGES 10 & 11	UM - 03	J. Austin	18/11/99		
PAGE 2	UM - 04 (DCN958)	J. Barker	30/10/00		

# 10. Technical Specification

Akron M6M				
	8462			
Head length (cm)	48±1.5			
Mid Length (cm)	47±1.5			
Foot Length (cm)	104±1.5			
Width (cm)	60±1.5			
Overall Length (cm)	203±3			
Height Range (cm)	62±1.5			
	109±1.5			
Max. Load Weight	180 Kg			
Elevation Control	Electric			
	Electrical Data 230v			
Actuator	220-240v			
Internal Fuse Rt	T1.6A x 1			
Current	1.6A			
Power	480W			
Mains Plug Fuse Rt (UK)	5.0A x 1			
	Electrical Data 115v			
Actuator	115v			
Internal Fuse Rt	F4.0A x 1			
Current	3.3A			
Power	380W			

# **IEC 601-1 Classification**

# Check serial number label for classification

	230v		115v		115v
	Double Insulated		Double Insulated		Class 1
<b>†</b>	Type B equipment	<b>†</b>	Type B equipment	<b>†</b>	Type B equipment
<u> </u>	Attention consult this manual	<u> </u>	Attention consult this manual	$\overline{\mathbb{A}}$	Attention consult this manual



Huntleigh Akron, 1 Farthing Road, Ipswich, Suffolk, IP1 5AP, England Tel: +44 (0) 1473 461042 Fax: +44 (0) 1473 462924